Plasma

Plasma
Foal IgG

A qualitative test
of immunity levels
to determine the
health and value
of newborn foals.

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The Plasma Foal IgG Midland Quick Test Kit[®] is designed to detect immunoglobulin G (IgG) concentrations from 0 to 7 day foals, in a rapid test format without the use of elaborate laboratory equipment. The kit is sold in two formats: a 4 mg/mL (400 mg/dL) and an 8 mg/mL (800 mg/dL) kit.

The rapid identification of foals with adequate IgG levels can be used to assess management and husbandry practices. In addition, if low IgG levels are detected, intervention strategies may be developed in conjunction with your health professional to optimize foal health and farm productivity.

Step1: Sample Collection

Contact your veterinarian for proper collection technique. Collection supplies are not included in the kit. **You will need:** collection tubes containing anticoagulant, blood collection needle and holder, or syringe and marking pen.

Obtain Sample

1. Label the collection tube with foal number, date, time of collection, etc.
2. Collect approximately 1 to 2 mL of whole blood using a collection tube and needle or using a syringe and expelling into the tube.
3. If clotting, centrifugation at 1,000 x g for 10 minutes will aid the separation of the sample from the clot.
Optional: Allow the blood sample to clot at room temperature for 30-90 minutes. Accelerate clotting by placing a wooden applicator stick into the collection tube with the blood. After clotting, the sample (which is the liquid portion) will be visible.
4. The sample may be stored refrigerated, in the collection tube (red top tube) with the clot, for 4-7 days. Clots should not be frozen. If sample is drawn off the clot, it may be stored in a tightly sealed container for weeks in a refrigerator or freezer.

Step 2: Procedure - See Separate Insert

Step 3: Interpretation of Test Results

Example of Plasma Foal IgG Midland Quick Test Kit[®] cassette results; Control = "C" Test = "T" Sample = "S"

In a properly functioning test, a line will appear at the Control (C) position of the results window.

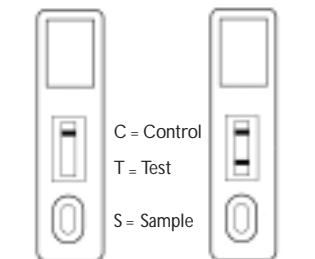
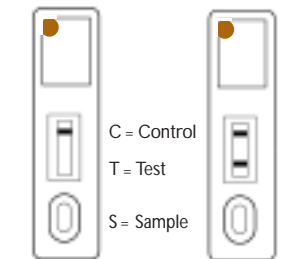
Adequate Level is indicated by ONE line at the "C" position.

Inadequate Level is indicated by TWO lines; One at the "T" position and One at the "C" position. Irregardless of the line intensity, a faint line at the "T" position should be interpreted as an **inadequate level** of IgG.

Note: Brown dot ● distinguishes the 4 mg/mL cassette from the 8 mg/mL cassette.

4 mg/mL Plasma Foal IgG 4 mg/mL Plasma Foal IgG

8 mg/mL Plasma Foal IgG 8 mg/mL Plasma Foal IgG

>4 mg/mL
(>400 mg/dL)
level of IgG
Adequate

<4 mg/mL
(<400 mg/dL)
level of IgG
Inadequate

>8 mg/mL
(>800 mg/dL)
level of IgG
Adequate

<8 mg/mL
(<800 mg/dL)
level of IgG
Inadequate

Contents of Plasma Foal IgG Midland Quick Test Kit[®]

This improved procedure utilizes a 200 microliter Pipetter which is sold separately (catalog no. 0019). Each kit contains a cassette, disposable pipette tips, dilution vial and instructions. Kits are packaged and sold in the following sizes:

4 mg/mL (400 mg/dL)

Package size	Catalog No.
Package of 6	42006
Package of 12	42012
Package of 24	42024

8 mg/mL (800 mg/dL)

Package size	Catalog No.
Package of 6	52006
Package of 12	52012
Package of 24	52024

Storage

- Avoid prolonged temperature and humidity extremes.
- Do not remove the cassette from the foil pouch until ready for use.
- Even though the foil pouch includes a desiccant packet, exposure to high humidity conditions should be minimized.
- If the buffer in the dilution vial freezes, thaw before using.

Precautions/Warnings

- Do not use components past expiration date.
- Do not use components from kits with different lot numbers.
- The dilution vial in this kit contains sodium azide. Sodium azide may react with lead or copper plumbing to form highly explosive metal azides. On disposal, flush with a large volume of water to prevent azide build-up.

For further information, refer to the manual issued by the Centers for Disease Control.

- Do not ingest desiccant or the buffer solution.
- Dispose of all kit components in an appropriate manner.

Bibliography

1. McGuire TC, Crawford TB, Hallowell AL, Macomber LE. Failure of colostral immunoglobulin transfer as an explanation for most infections and deaths in neonatal foals. *JAVMA* 170(June 1, 1977) : 1302-1304.
2. LeBlanc MM, McLaurin BI, Boswell R. Relationships among serum immunoglobulin concentration in foals, colostral specific gravity and colostral immunoglobulin concentration. *JVMA* 189 (July 1, 1986) :57-60.
3. Dwyer RM. *The Horse*, (Sept 1999) :106.

Other Midland Quick Test Kits® available or in development:

Whole Blood Calf IgG

Catalog No. 24006, 24012 and 24024

Yes/No test with a cut off of 10 mg/mL (1000 mg/dL) of IgG

Plasma Calf IgG

Catalog No. 14006, 14012 and 14024

Yes/No test with a cut off of 10 mg/mL (1000 mg/dL) of IgG

Whole Blood Foal IgG 4mg/mL or 400 mg/dL

Catalog No. 62006, 62012 and 62024

Yes/No test with a cut off of 4 mg/mL (400 mg/dL) of IgG

Whole Blood Foal IgG 8mg/mL or 800 mg/dL

Catalog No. 72006, 72012 and 72024

Yes/No test with a cut off of 8 mg/mL (800 mg/dL) of IgG

Colostrum Bovine IgG

Catalog No. 34006, 34012 and 34024

Yes/No test with a cut off of 50 mg/mL (5000 mg/dL) of IgG

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Bibliographie

1. McGuire TC, Crawford TB, Hallowell AL, Macomber LE. Failure of colostral immunoglobulin transfer as an explanation for most infections and deaths in neonatal foals. JAVMA 170(June 1, 1977) : 1302-1304.
2. LeBlanc MM, McLaurin BI, Boswell R. Relationships among serum immunoglobulin concentration in foals, colostral specific gravity and colostral immunoglobulin concentration. JAMA 189 (July 1, 1986):57-60.
3. Dwyer RM. The Horse, (Sept 1999) :106.

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1. McGuire TC, Crawford TB, Hallowell AL, Macomber LE. Failure of colostral immunoglobulin transfer as an explanation for most infections and deaths in neonatal foals. JAVMA 170(June 1, 1977) : 1302-1304.
2. LeBlanc MM, McLaurin BI, Boswell R. Relationships among serum immunoglobulin concentration in foals, colostral specific gravity and colostral immunoglobulin concentration. JVMA 189 (July 1, 1986) :57-60.
3. Dwyer RM. The Horse, (Sept 1999) :106.

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Bibliografie

1. McGuire TC, Crawford TB, Hallowell AL, Macomber LE. Failure of colostral immunoglobulin transfer as an explanation for most infections and deaths in neonatal foals. JAVMA 170 (June 1, 1977) : 1302-1304.
2. LeBlanc MM, McLaurin BI, Boswell R. Relationships among serum immunoglobulin concentration in foals, colostral specific gravity and colostral immunoglobulin concentration. JVMA 189 (July 1, 1986) :57-60.
3. Dwyer RM. The Horse, (Sept 1999) :106.

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